




# TEMPORARY PROSTHESIS

■ Achievement of a simplified immediate temporary prosthesis aesthetic and comfortable

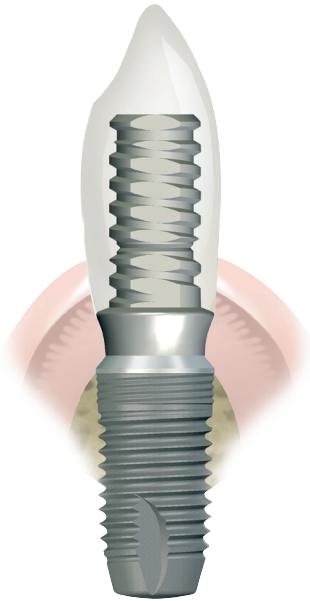

- Facilitates the temporizing
- Promotes soft tissue healing
- Provides guidance for gingival healing during bone maturation = improvement of the final prosthesis aesthetic
- Used to validate the treatment (aesthetic, phonetic smile)
- A single abutment for all implants diameters with the possibility to customize its height

Ref. UPS PT 41 100

## PROTOCOLS

### Achievement of the prosthesis in under-occlusion immediately after implant placement

- Fitting of the abutment in the mouth
- If necessary the abutment can be cut to get the desired height.
- Fixation of the abutment in mouth directly on the implant to 35N.cm with the titanium TA6V screw delivered with the abutment.
- Make the composite directly on the provisional abutment.

### Early loading

- Impression with the pick-up impression coping (ref UPS TCO 50) screwed on the implant head.
- Connecting the analog to the transfer to make the stone model.
- Fixation of the temporary abutment on the analog which can be cut if necessary to get the desired height.
- Achievement of cosmetic in the laboratory.
- Fixation of the abutment in mouth directly on the implant at 20N.cm with the titanium TA6V screw delivered with the abutment.

